



# DAFS Express

Installation and License Activation

Voronoi Health Analytics Inc.

# Table of Contents


Page Number

<a href="#">Setup and Installation</a>	3
<a href="#">License Request and Activation</a>	5
<a href="#">FAQs</a>	11



# Setup and Installation

1. Download MATLAB Runtime version 2023a (R2023a (9.14)) [here](#). After downloading, unzip the file, and open the setup.exe. Complete installation according to the setup instructions. Please note that download and installation can take several minutes depending on your hardware specifications.

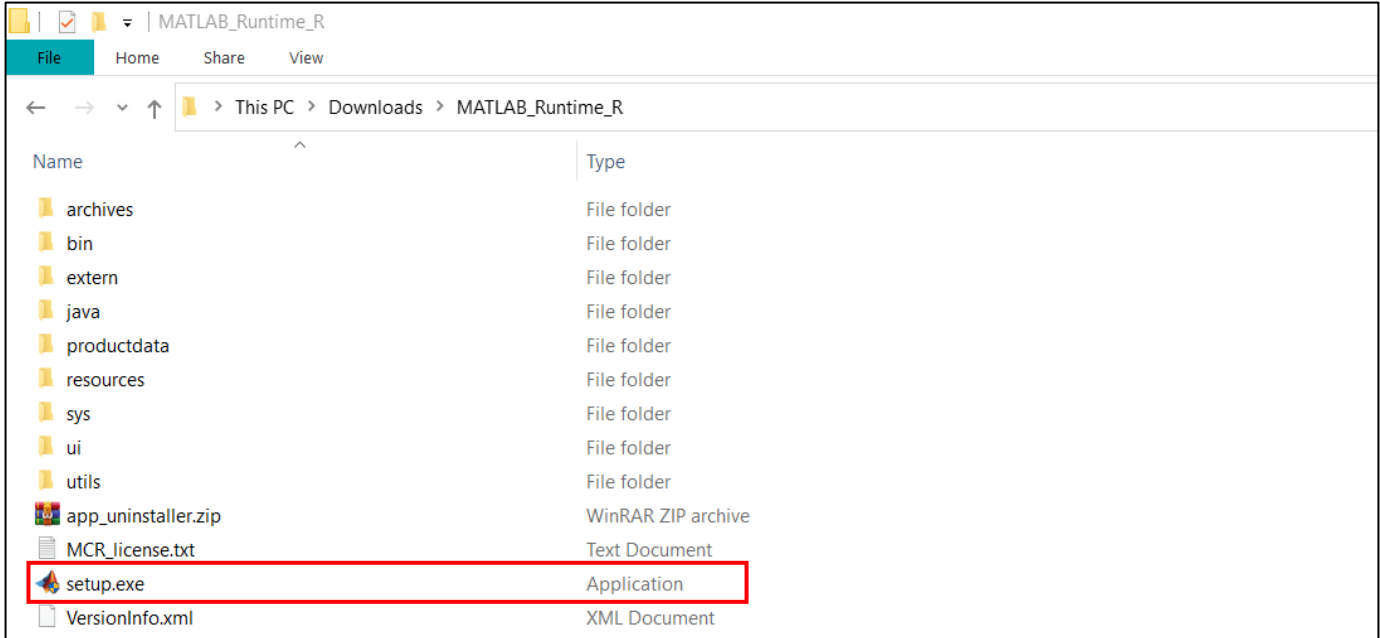


2. Next, click [here](#) and download DAFS Express-v1.x.x\_installer.exe. Complete installation according to the setup instructions and make sure to click the “Create a desktop shortcut” option during installation.




# License Request and Activation


1. Insert the USB dongle provided to you into your computer.

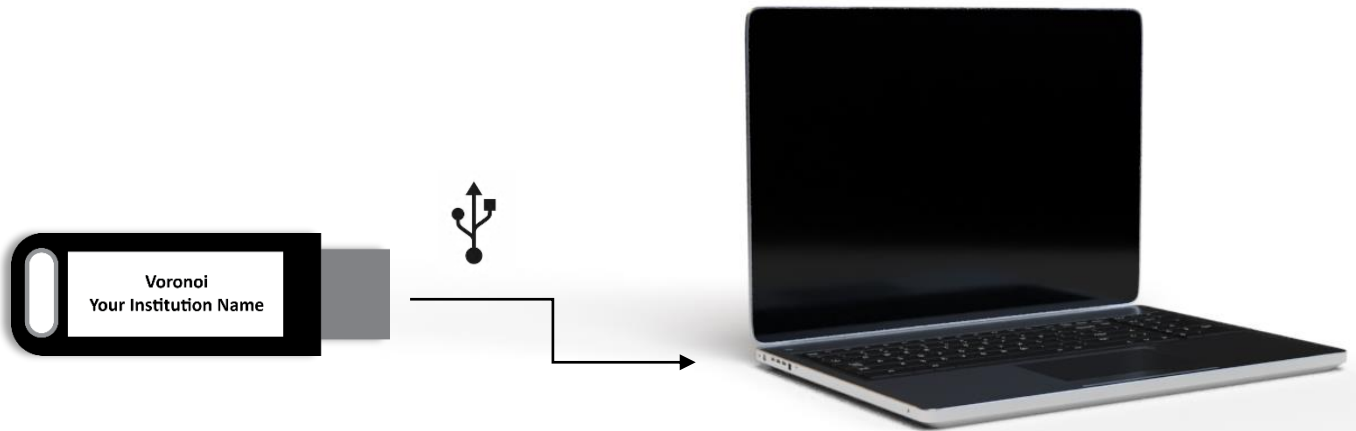


2. Click [here](#) to download a [Voronoi\\_License\\_Activator.exe](#) (a .zip is available if needed).

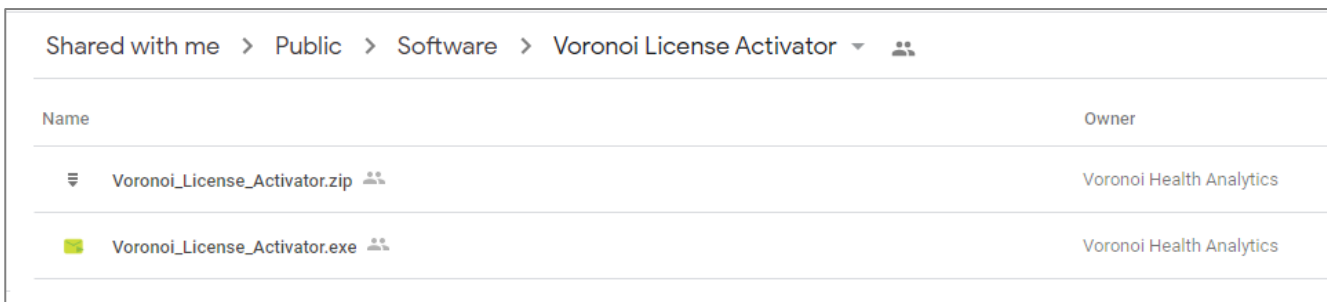

3. Ensure “Update of existing protection key” is selected and click Collect Information and save the .c2v to a location on your local computer (do not select “Installation of new protection key”).

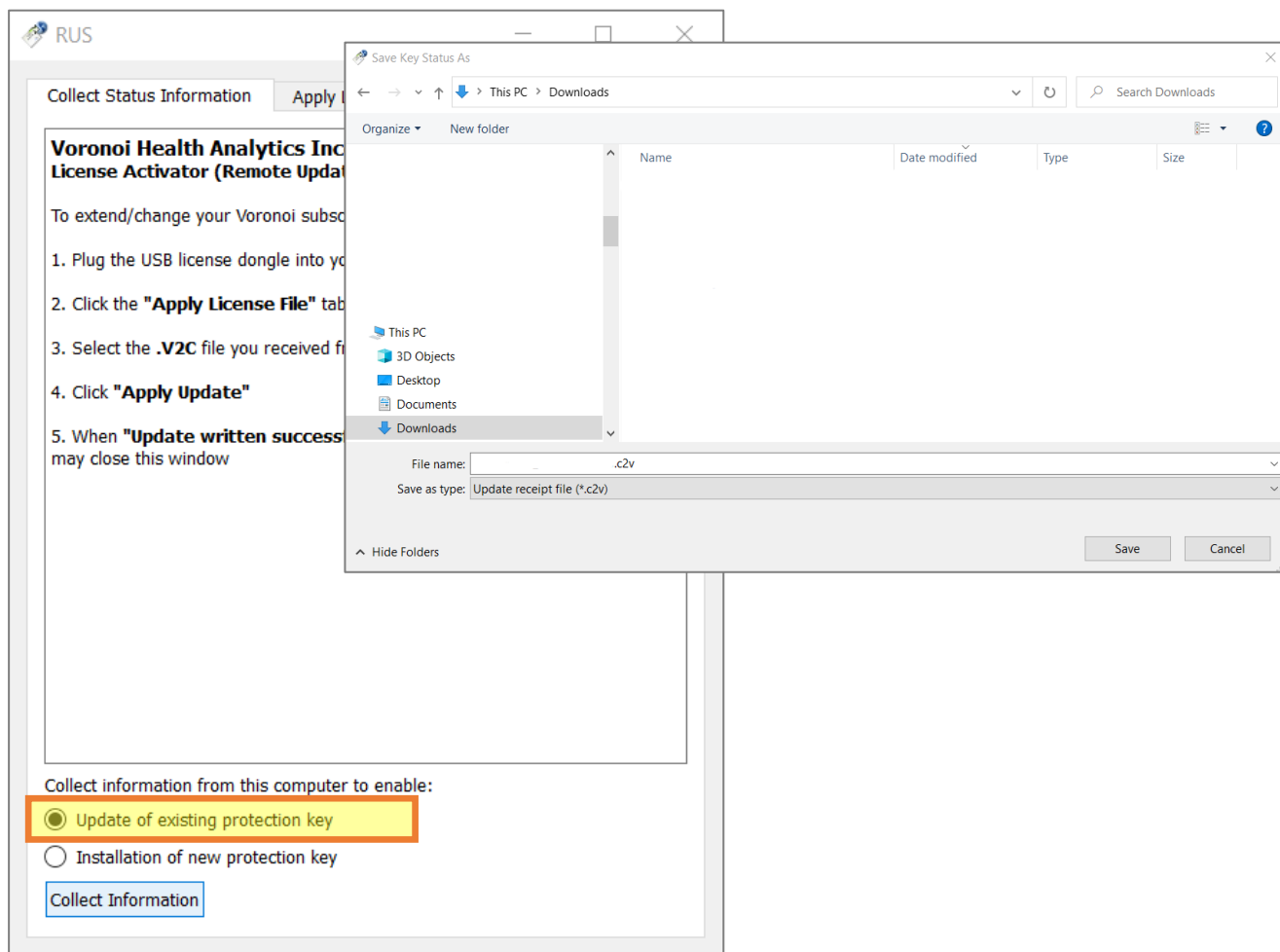

3. Email the .c2v file attached to [support@voronoihealthanalytics.com](mailto:support@voronoihealthanalytics.com).

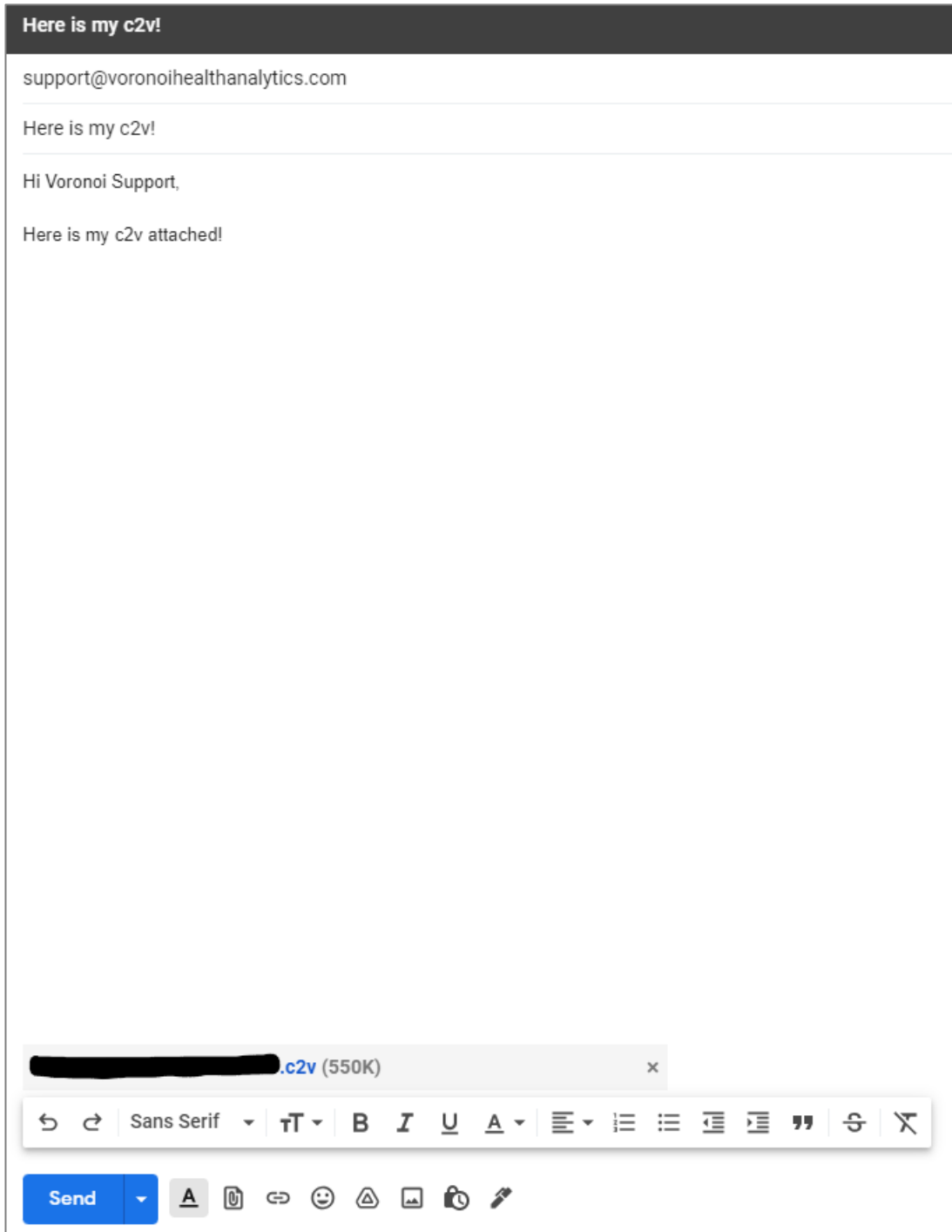


4. Our team will be hard at work preparing your license package as soon as we receive this your .c2v file!






7. Once the .V2C has been successfully applied to the dongle, the following message will appear. Close the Voronoi License Activator.

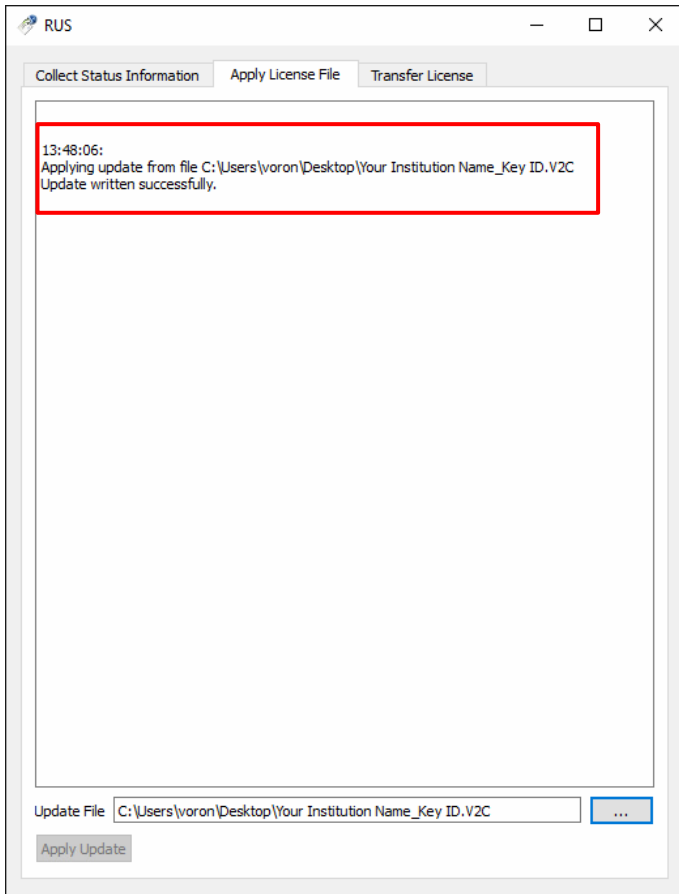


8. In your license file package folder, click on the VHAIm.exe and allow the license to update completely until you see a confirmation pop-up.

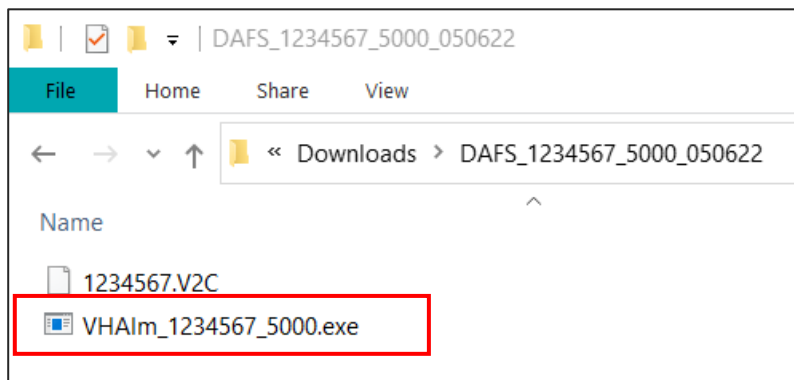



9. You are now ready to start using DAFS Express! When you open DAFS Express, you will see the following splash screen. Please be patient as the DAFS window may take a few moments to open after the splash screen disappears.

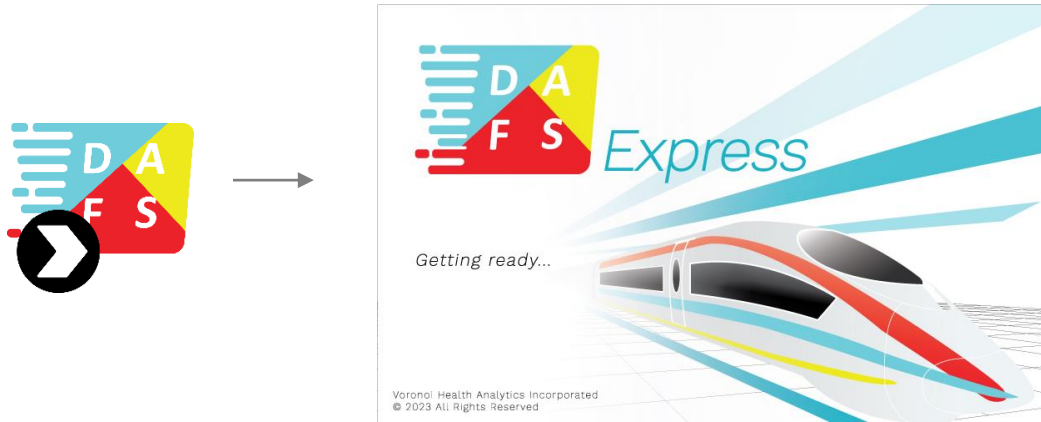


DAFS Express requires the USB dongle to be inserted into your computer at all times while in use.




# FAQs

- I am getting a 'Sentinel key not found' error when I try to open DAFS Express. What is happening how do I fix this?

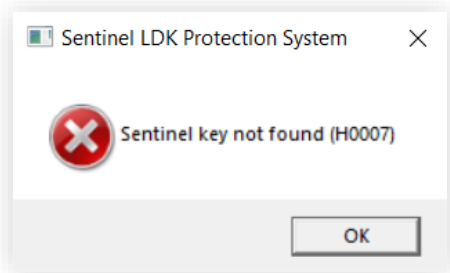

If you are seeing this error, your DAFS Express USB is likely not plugged in. Ensure that your USB is properly connected and then click 'OK'.

- I am getting a 'Feature not found' error when I try to open DAFS Express. What is happening and how do I fix this?

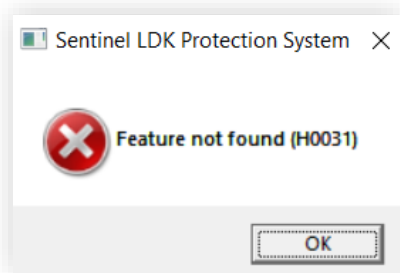

If you are seeing this error, you must use your license package files (.V2C and VHAlm.exe) to activate your license.

See page 4 for instructions.

- How can I find out when my DAFS Express license expires?

The license expiry date and other important information relating to your DAFS Express license can be found in the license and information button in DAFS Express.

If your DAFS Express subscription has expired and you wish to renew it, please contact us at [support@voronoihealthanalytics.com](mailto:support@voronoihealthanalytics.com) to get more information on how to proceed.

- Can I run DAFS Express over an RDP (Remote Desktop Protocol) connection?

Yes, DAFS Express supports running over Remote Desktop Protocol.

- I am still having problems with installation/license activation. Who do I contact?

Please contact our team at [support@voronoihealthanalytics.com](mailto:support@voronoihealthanalytics.com) if you require further assistance with installation, license activation, renewals, or anything else contained in this instructional guide.

